**Instructions for use**

23331762 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2024-03-01

**Intended purpose**

The ostomy bag is intended to passively collect output from a stoma. The adhesive baseplate is attached to the ostomy bag and is intended to adhere to intact skin around the stoma.

**Indications**

The device is indicated for people with a stoma.

**Warnings**

Re-use of the single use product is not recommended as cross contamination may occur, potentially leading to local infection.

Reprocessing, washing, disinfection and/or (re)sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

**Cautions**

For appropriate use, consult your health care professional before you start using a convex product.

**Information**

Single use ostomy bag.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

The device does not have to be removed before an MR scan as it is MR safe.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.


**Special storage conditions**

Store product horizontally.


Keep away from sunlight.

**How to use**

**Preparation**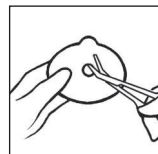

Cut a hole in the baseplate to fit the exact size and shape of the stoma using a pair of small curved scissors and the cutting guide on the baseplate. You can use the enclosed stoma measuring guide to measure the size and shape of the stoma. An exact fit between the hole and the stoma is important to reduce the risk of leakage and skin problems.

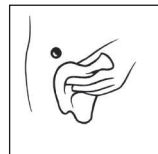
Thoroughly clean the stoma and the surrounding skin area. The skin must be completely dry before application of the baseplate.

**Application**

Peel off the protective film from the baseplate.

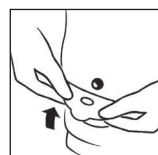
Align the baseplate around the stoma and apply pressure to the baseplate to ensure fast adhesion and a perfect fit between the adhesive and the skin around the stoma. The adhesive must fit snugly around the stoma. To prevent leakage it is important to ensure a tight seal around the stoma.

When applying pressure to the baseplate/bag on the abdomen, begin in the area around the stoma and lightly press the adhesive to the skin. Run a finger all the way around the stoma to ensure that the adhesive seals. Smooth the adhesive down, starting from the stoma and moving outwards to the edges, to ensure that the adhesive has made full contact with the skin.

For information on how to use the bag, please refer to the instruction for use that accompany the retail box for the bags.

**Removal**

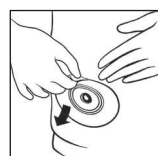Gently remove the bag by pulling the removal ear on the adhesive away from the skin and roll the adhesive slowly downwards. Apply light pressure on your skin with your free hand to help removal.

**Disposal**

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

**Reporting of incidents**

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 5 pieces per retail box

## Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Indicates a carrier that contains Unique Device Identifier information

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