**Instructions for use**

Please read the following instructions carefully.

23327872 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-08-06

**Intended purpose**

The ostomy bag is intended to passively collect output from a stoma. The adhesive baseplate is attached to the ostomy bag and is intended to adhere to intact skin around the stoma.

**Indications**

The device is indicated for people with a stoma.

**Warnings**

Re-use of the single use product is not recommended as cross contamination may occur, potentially leading to local infection.

Reprocessing, washing, disinfection and/or (re)sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

**Information**

The Post Op bag is a non-sterile single use product.

The device does not have to be removed before an MR scan as it is MR safe.


Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

**Special storage conditions**

Keep away from sunlight.

Store product horizontally.

**Preparation**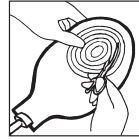


Cut a hole in the adhesive to fit the exact size and shape of the stoma using a pair of small curved scissors and the cutting guide on the adhesive.

You can use the enclosed stoma measuring guide to measure the size and shape of the stoma.


An exact fit between the hole and the stoma is important to reduce the risk of leakage and skin problems.

Thoroughly clean the stoma and the surrounding skin area. The skin must be completely dry before application of the adhesive.

**Application**

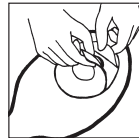
Peel off the protective film from the adhesive.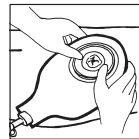

Align the adhesive around the stoma and press it firmly onto the skin.

The adhesive must fit snugly around the stoma. To prevent leakage it is important to ensure a tight seal around the stoma. When applying pressure to the baseplate/bag on the abdomen, begin in the area around the stoma and press the adhesive to the skin. Run a finger all the way around the stoma to ensure that the adhesive seals. Smooth the adhesive down, starting from the stoma and moving outwards to the edges, to ensure that the adhesive has made full contact with the skin.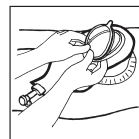


Only for bags with inspection window

Place your fingers under the flexible flange/ring and close the window by moving your fingers to the top of the coupling. The window will close with a click.

The window is removed from the bag by holding the small tab on the window while the other hand holds the bag.

**Emptying/Drainage**

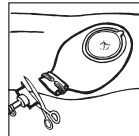Hold the plug/end of the outlet upwards and squeeze the outer end of the outlet, pull out the plug, and relax your hold on the outlet and then drain the bag. After complete drainage, ensure that the outlet is firmly closed with the plug fully inserted.

The bag is supplied with an outlet which can be used in connection with a drainage bag. The outlet tap can also be cut off, and the outlet can be closed by using a clamp.

**Removal**

Gently remove the bag by pulling the removal ear on the adhesive away from the skin and roll the adhesive slowly downwards.

Apply light pressure on your skin with your free hand to help removal.

**Disposal**

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

**Reporting of incidents**

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 6 pieces per retail box

### Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Non-sterile

Indicates a carrier that contains Unique Device Identifier information

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