Brava®

 **Coloplast**

Brava Protective Sheet

Coloplast A/S
Høttedam 1
3050 Humlebaek
Denmark

Instructions for use

23331765 Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2024-04-11

Intended purpose

Intended for protection of intact skin against stoma output, and for skin protection where body secretion may damage the skin.

Warnings

Re-use of the single use product may create a potential harm to the user.

Reprocessing, washing, disinfection and/or sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

Information

The sheet is highly adhesive in order to provide the greatest possible security, and it is very important that all hair is removed from the application site before the sheet is applied.

Both adhesion and security are reduced if cream is used. However, a thin layer of cream can be used in certain cases if you find the Protective Sheet adheres too strongly.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.


How to use


Remove the protective paper from the non-adhesive side of the sheet (bend at split-line and remove protective paper).


Mark the desired hole size with a pen.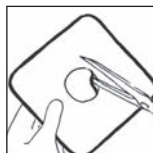

With a pair of scissors, cut out the desired hole size according to the marking.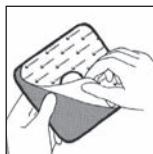

Clean and dry skin around the stoma.
Remove the protective paper from the adhesive side.

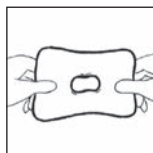Place the adhesive on the skin and press firmly with your fingers. The elasticity of the sheet allows the hole to widen when it is put in place e.g. over a stoma. It retracts immediately ensuring a completely snug fit around the stoma.

Removal

The Protective Sheet should be removed slowly and carefully.

Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 10 pieces per retail box

Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Indicates a barcode as containing Unique Device Identification

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