# SenSura®

# Coloplast

SenSura Click Ostomy bag  
2-piece open

Ostomy baseplate

Ostomy Care

### Instructions for use

23331770 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2024-03-06

Velcro® is owned by Velcro Industries B.V

### Intended purpose

The ostomy bag is intended to passively collect output from a stoma. The adhesive baseplate is attached to the ostomy bag and is intended to adhere to intact skin around the stoma.

### Indications

The device is indicated for people with a stoma.

### Warnings

Re-use of the single use product is not recommended as cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

### Cautions

For appropriate use, consult your health care professional before you start using a convex product.

### Information

Single use ostomy bag.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

The device does not have to be removed before an MR scan as it is MR safe.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.


### Special storage conditions

Store product horizontally.

Keep away from sunlight.

## How to use


### Preparation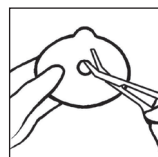

Cut a hole in the baseplate to fit the exact size and shape of the stoma using a pair of small curved scissors and the cutting guide on the baseplate.


You can use the enclosed stoma measuring guide to measure the size and shape of the stoma.

An exact fit between the hole and the stoma is important to reduce the risk of leakage and skin problems.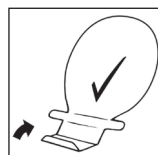


Place the bag on a flat surface with the coupling side down.

Close the outlet by folding the bottom of the outlet upwards three times so that the Velcro® strip becomes visible.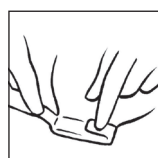

Fold the two tabs inwards to secure the Velcro®.

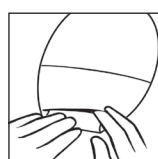The outlet can be hidden away by using either the Velcro® dot immediately above the outlet, or by using the Velcro® strip on the middle of the bag.


(This does not apply to the transparent bags where there is no Velcro® strip on the middle of the bag).

Thoroughly clean the stoma and the surrounding skin area. The skin must be completely dry before application of the baseplate.

### Application


Peel off the protective film from the baseplate.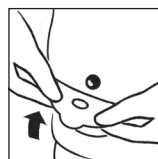


Align the baseplate around the stoma and apply pressure to the baseplate to ensure fast adhesion and a perfect fit between the adhesive and the skin around the stoma.

The adhesive must fit snugly around the stoma. To prevent leakage it is important to ensure a tight seal around the stoma. When applying pressure to the baseplate/bag on the abdomen, begin in the area around the stoma and press the adhesive to the skin. Run a finger all the way around the stoma to ensure that the adhesive seals. Smooth the adhesive down, starting from the stoma and moving outwards to the edges, to ensure that the adhesive has made full contact with the skin.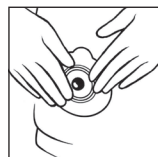

Ensure the baseplate is dry and clean (when cleaning the baseplate only use water).

Make sure that the lock ring is open. Position the bottom of the bag coupling at the bottom of the baseplate coupling.

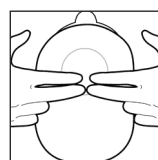Press the bag and the baseplate together at the bottom with your middle fingers.


With your index fingers apply a gentle pressure to the top of the coupling until you feel the bag is correctly positioned to the baseplate.

Close the lock ring by pressing the lock together. The bag is securely attached to the baseplate when you hear a 'click'.

#### How to empty the bag

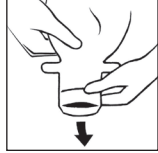Open the outlet and empty the bag into the toilet when needed.


Clean the outlet, for example with toilet paper. Fold and resecure the outlet after you have emptied the bag.


#### Removal

Remove the bag by opening the lock ring by pressing the small dotted tab facing your stomach with a finger tip.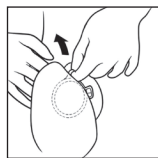

Hold on to the removal ear and remove the bag by carefully pulling the bag away from the baseplate.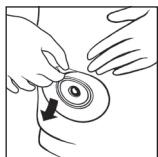

Gently remove the baseplate by pulling the removal ear on the baseplate away from the skin and roll the baseplate slowly downwards. Apply light pressure on your skin with your free hand to help removal.

#### Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.


Do not flush the product down the toilet.

#### Filter label

If required, to prevent gas from escaping, place one of the enclosed filter labels on the filter.

#### Belt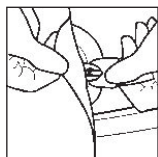

Adjust the belt length by pulling the straps, so it fits your body. There is no need to remove the fabric covering the buckle.

Insert the hook of the belt into the belt ear on the ostomy bag or baseplate. Then wrap the belt around the body and fasten the hook on the other end of the belt to the other belt ear on the ostomy bag or baseplate.

If necessary, adjust the length of the belt.  
(Only applies to products with belt ears).

#### Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 30 pieces per retail box

#### Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Indicates a carrier that contains Unique Device Identifier information

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