**Brava**<sup>®</sup>

## Brava Adhesive Remover Wipe

### Instructions for use

23327871 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-07-27

### Intended purpose

Brava Adhesive Remover wipe is intended for removal of medical adhesives from intact skin.

### Warnings

Re-use of the single use product is not recommended cross contamination may occur.

Reprocessing, washing, disinfection and/or (re) sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

### Cautions

Avoid direct use on non-intact skin e.g. wound beds and mucous membranes (eyes, ears, nose, mouth, vagina) as it may cause skin irritation.

Not recommended to be used on premature infants, neonates or infants below 4 kg because of their immature skin.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

### Information

The product is silicone based. The product dries within seconds and does not have an effect on the adhesion of further appliances.

Ingredients: Contains a mixture of alkyl siloxanes.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

### How to use

1. Tear sachet to open and take out the tissue and unfold it.

2. Wipe away residues.

### Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

### Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 30 pieces per retail box

### Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Indicates a carrier that contains Unique Device Identifier information

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