Brava[®]

Brava Belt

Belt for support of ostomy bags and plates

Instructions for use
23327848 Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-11-24

Intended purpose

The Brava Belt is intended for support of ostomy bag and plate by holding the ostomy bag and plate in the correct position.

Information

The Brava Belt can be washed at 60°C.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use

Adjust the belt length by pulling the straps, so it fits your body. There is no need to remove the fabric covering the buckle.

Insert the hook of the belt into the belt ear on the ostomy bag or baseplate. Then wrap the belt around the body and fasten the hook on the other end of the belt to the other belt ear on the ostomy bag or baseplate.

If necessary, adjust the length of the belt.

Disposal

The product should be disposed of with normal household waste or according to local guidelines.

Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 15 pieces per retail box

Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Indicates a barcode as containing Unique Device Identification

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