# Brava®

Brava Elastic Tape  
Elastic Tape XL  
Elastic Tape Belt  
Elastic Tape Straight

Coloplast A/S  
Høtveddam 1  
3050 Humlebaek  
Denmark

## Instructions for use

23327870 Version 1  
The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. ©  
2021-07-26

## Intended purpose

Brava Elastic Tape is intended to support the rim of the ostomy baseplate/adhesive by providing additional adhesion to the intact peristomal skin.

## Indications

Brava Elastic Tape is indicated for people who use an ostomy bag and baseplate/adhesive.

## Warnings

Re-use of the single use product is not recommended cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

## Cautions

Keep away from sunlight as it may impact product quality.

Do not stretch the product when applying as it may impact product performance.

## Information

Single use product.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

The device does not have to be removed before an MR scan as it is MR safe.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

## Special storage conditions

Keep away from sunlight.

Store product horizontally.

## How to use


### Opening the box

Hold the turquoise tab and pull back to open. Do not tear the tab as it allows you to re-seal the box.

### Application

#### Elastic Tape


#### Elastic Tape XL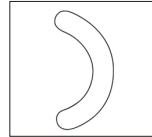


Remove either of the protective paper sheets before applying.

Place the tape where desired on the baseplate/adhesive where you need support. Remember that a part of the adhesive should be covered by the tape.

Do not stretch the product when applying.

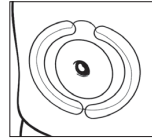
Remove the remaining protective paper sheet and smooth the tape onto the skin. Repeat if necessary.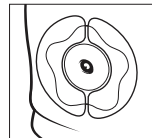

Elastic Tape XL can also be used with SenSura® Mio Concave baseplate.


#### Elastic Tape Belt

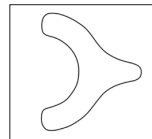Remove either of the protective paper sheets before applying.


Place the tape where desired on the baseplate/adhesive where you need the extra support. Remember that a part of the adhesive should be covered by the tape.

Do not stretch the product when applying.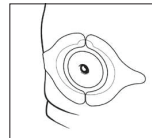

Remove the remaining protective paper sheet and smooth the tape onto the skin. Repeat if necessary.


#### Elastic Tape Straight

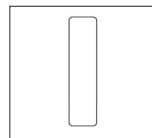Remove either of the protective paper sheets before applying.

Place the tape where desired on the square baseplate/adhesive where you need support. Remember that a part of the adhesive should be covered by the tape.

Do not stretch the product when applying.

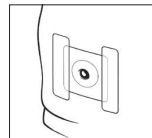Remove the remaining protective paper sheet and smooth the tape onto the skin. Repeat if necessary.

You can place the tape wherever you feel you need to support the baseplate/adhesive.

### Removal of the tape

The tape may come off together with the adhesive. If needed, loosen the edges of the tape and gently stretch the tape for removal.

### Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

### Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 20 pieces per retail box

### Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

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