Brava[®]

en Brava Mouldable Ring

Fills in cavities/folds

Ostomy Care

Instructions for use

Please read the following instructions carefully.

23327864 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-07-26

Intended purpose

The product is intended to fill in cavities/folds in the intact skin around the stoma. This provides a seal between the stoma and the ostomy baseplate and protects the skin against the stoma output.

Warnings

Re-use of the single use product is not recommended as cross contamination may occur, potentially leading to local infection.

Reprocessing, washing, disinfection and/or sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

Cautions

This product is non-sterile and should be used with caution on premature babies or infants receiving incubator care as it may lead to increased risks of infection.

For external use only.


Keep away from sunlight.

Information


Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use

Preparation

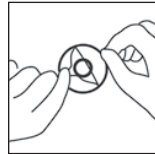
Before applying the Mouldable Ring, ensure that the skin around the stoma is clean and dry. Remove the ring from the blister, and then peel off the protective paper from both sides by holding the removal ears.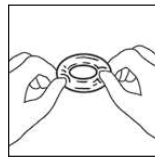


Mould and stretch the ring to the desired shape.


Application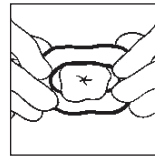

Apply the ring to the skin around the stoma. Adjust the shape again if needed. Mould the paste with your fingers so that the edge of the ring is securely smoothed onto the skin.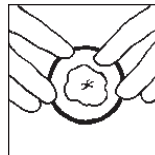

If you use a 2-piece ostomy appliance, it is possible to apply the ring directly to the coupling side of the baseplate, to achieve a snug fit around the stoma.

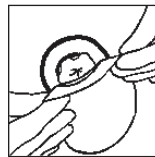Apply the bag as normal.

Removal

After usage, gently remove the Mouldable Ring from the skin by rolling it step by step downwards. The Mouldable Ring should come off together with the adhesive. However, if there are slight residues left on the skin it does not damage the skin or have an effect on adhesion of the next Mouldable Ring.

Disposal

The product should be disposed of with normal household waste or according to local guidelines.

Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 30 pieces per retail box

Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Indicates a carrier that contains Unique Device Identifier information

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