**Brava**<sup>®</sup>

 **Coloplast**

**en** Brava Paste  
Paste

Fills in cavities/folds

Ostomy Care

**Instructions for use**

Please read the following instructions carefully.

23327865 Version 3

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-12-09

Imported by:  
DKSH (Thailand) Limited  
2106 Sukhumvit Road, Phrakhanong-Tai,  
Phrakhanong, Bangkok 10260 Thailand  
Tel: (66) 2220 9000

**Intended purpose**

The product is intended to fill in cavities/folds in the intact skin around the stoma. This provides a seal between the stoma and the ostomy baseplate and protects the skin against the stoma output.

**Cautions**

This product is non-sterile and should be used with caution on premature babies or infants receiving incubator care as it may lead to increased risks of infection.

For external use only.

Keep away from sunlight.

**Information**

If the paste separates, the watery part should be discarded.

Discard the tube if the paste turns dark.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

**How to use**

**Preparation**

In order to avoid contamination of the paste


- Keep the tube away from water and prevent contact between the tube and stoma.
- Wash your hands with soap and water before and after applying the paste.

Ensure that the skin around the stoma is clean and dry before applying the paste.


**Application**

Apply the paste onto the skin to level deep skin folds, scars or uneven surfaces or apply directly on the adhesive/baseplate. Do not apply paste on the entire adhesive area.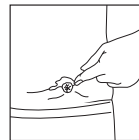

If needed, level the paste by gently smoothing using a spatula or finger that has been moistened.

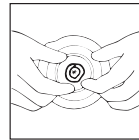After applying the baseplate/adhesive, ensure a perfect fit around the stoma by pressing on the entire baseplate/adhesive with your fingers.

After use remove the residual paste on the tube with a clean tissue. Recap the tube tightly to prevent contact with moisture.

**Removal**

The paste can easily be removed with lukewarm water and a wash cloth. An oily residue of paste can be left on the skin when applying the next appliance. It is better to leave a slight residue than risk irritating the skin through complete removal.

**Disposal**

The product should be disposed of with normal household waste or according to local guidelines.

Do not flush the product down the toilet.

**Reporting of incidents**

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 1 piece per retail box  
60g

### Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Caution

Indicates a carrier that contains Unique Device Identifier information

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