Brava[®]

en Brava Powder
Powder

Ostomy Care

Instructions for use

Please read the following instructions carefully.

23327849 Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved.© 2021-09-21

Imported by:
DKSH (Thailand) Limited
2106 Sukhumvit Road, Phrakhanong-Tai,
Phrakhanong, Bangkok 10260 Thailand
Tel: (66) 2220 9000

Intended purpose

Powder is intended for moisture absorption on intact peristomal skin.

Cautions

This product is non-sterile and should be used with caution on premature babies or infants receiving incubator care as it may lead to increased risks of infection.

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Special storage conditions

The powder should be stored away from direct sunlight, in a cool and dry place.

How to use

Preparation

1. Wash and dry the skin around the stoma.

Application

2. Open the lid and aim at the intended area.
3. Shake or gently squeeze the bottle and spread the powder on the intended intact skin.
4. Spread the powder evenly using gauze.
5. Dust off excess powder before applying the ostomy baseplate or ostomy bag adhesive.
6. Close the lid tightly after use and store in a cool, dry place.

Disposal

The product should be disposed of with normal household waste or according to local guidelines. Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 16 pieces per retail box
25g

Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Indicates a carrier that contains Unique Device Identifier information

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