Brava®

Brava Skin Barrier Spray
Skin Barrier Spray

Instructions for use
23327863 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-07-26

Intended purpose

The product is intended for protection of intact skin from the damaging effects of body waste and medical adhesives.

Cautions

Avoid direct use on non-intact skin e.g. wound beds and mucous membranes (eyes, ears, nose, mouth, vagina) as it may cause skin irritation.

Avoid direct inhalation as it may give rise to temporary irritation of the airways.

Not recommended to be used on premature infants, neonates or infants below 4 kg because of their immature skin.

Highly flammable.

Keep out of reach of children.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Do not spray on an open flame or other ignition source.

Do not pierce or burn, even after use.

Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F.

Information

The product is silicone based and reusable. The product dries within seconds and does not have an effect on the adhesion of further appliances.

Ingredients: Contains a mixture of alkyl siloxanes.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use

1. Ensure that the skin is clean and dry.
2. Spray the product evenly on the intact skin from a distance of approximately 10 cm (cover non-intact skin).
3. Once the skin is completely dry, the appliance can be applied as usual.

A new application of the product is required at every appliance change.

Disposal

The product should be disposed of with other spray cans according to local regulations.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 1 pieces per retail box
50ml

Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalogue number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Indicates the upper limit of temperature to which the medical device can be safely exposed

Caution

Indicates a barcode as containing Unique Device Identification

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