Brava[®]

Brava Adhesive Remover Spray

Instructions for use

23327862 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-07-26

Intended purpose

Brava Adhesive Remover spray is intended for removal of medical adhesives from intact skin.

Cautions

Avoid direct use on non-intact skin e.g. wound beds and mucous membranes (eyes, ears, nose, mouth, vagina) as it may cause skin irritation.

Avoid direct inhalation as it may give rise to temporary irritation of the airways.

Not recommended to be used on premature infants, neonates or infants below 4 kg because of their immature skin.

Extremely flammable aerosol - Danger

Pressurised container: May burst if heated.

Keep out of reach of children.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Do not spray on an open flame or other ignition source.

Do not pierce or burn, even after use.

Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F.

Information

The product is silicone based and reusable. The product dries within seconds and does not have an effect on the adhesion of further appliances.

Ingredients: Contains a mixture of alkyl siloxanes.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use

1. Spray the product around the edge of the adhesive.

2. After a few seconds gently remove the adhesive. Repeat procedure if required.

Disposal

The product should be disposed of with other spray cans according to local regulations.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Pack size: 1 piece per retail box
50 ml

Explanation of symbols

	Medical device
	Indicates that the product is in compliance with European legislation for medical devices
	Catalogue number
	Use-by date (YYYY-MM-DD)
	Batch code
	Date of manufacture (YYYY-MM-DD)
	Manufacturer
	Consult instructions for use
	Indicates the upper limit of temperature to which the medical device can be safely exposed
	Caution
	Indicates a barcode as containing Unique Device Identification
	Global Trade Item Number
	Keep away from sunlight
	Recyclable packaging
	Compliance with Aerosol Dispenser Directive 75/324/EEC
	Extremely flammable

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